

# Servoport 3000

## Filter mit Silikonschläuchen

Ersatzteil für den Servoport 3000



### Bakterienfilter (mit Schlauchverbindung)

- Verbinden Sie den mit ‚IN‘ gekennzeichneten Einlass des Bakterienfilters mit dem Anschluss ‚OUT‘ auf dem Deckel des Sekretbehälters.
- Der Bakterienfilter muss bei jedem Patientenwechsel ausgetauscht werden. Wird das Gerät ausschließlich an einem Patienten betrieben, muss der Filter alle 2-3 Monate gewechselt werden.
- Beim Entfernen und Entleeren des Sekretbehälters darauf achten, dass keine Flüssigkeit in den Bakterienfilter läuft, da ein feuchter/nasser Filter die Absaugleistung verringert. In diesem Fall muss der Bakterienfilter gewechselt werden.

Gebrauchsanweisung des Servoport 3000 beachten!

**REF 26579**



**Orbisana Healthcare GmbH**  
Biberweg 24-26  
53842 Troisdorf  
Germany  
[www.orbisana.de](http://www.orbisana.de)

**EN**

## Servoport 3000 Filter with tubing connections

Spare part for Servoport 3000

### **Bacterial filter** (with tubing connection)

- Connect the inlet of the bacterial filter marked with 'IN' with the 'OUT' connection on the lid of the secretion container.
- The bacterial filter has to be replaced for each patient. If the apparatus is operated exclusively with one patient, the filter has to be replaced every 2-3 months.
- Take care when removing and emptying the secretion container that no fluid runs into the bacterial filter, because a moist/wet filter reduces the suction effect. In this case the bacterial filter has to be replaced.

Follow the instructions for use of the Servoport 3000!

**FR**

## Servoport 3000 Filtre à tubes en silicone

Pièce de rechange pour le Servoport 3000

### **Filtre antibactérien** (avec liaison par tube)

- Reliez l'orifice d'admission marqué « IN » du filtre antibactérien à l'orifice de sortie marqué « OUT » sur le couvercle du récipient à sécrétions.
- Le filtre antibactérien doit être remplacé lors de chaque changement de patient. Si l'appareil est toujours utilisé sur le même patient, le filtre doit être remplacé tous les 2-3 mois.
- Au moment de retirer et de vider le récipient à sécrétions, veiller à ce qu'il ne coule pas de liquide dans le filtre antibactérien vu que le filtre humide/mouillé réduit la puissance d'aspiration. Dans ce cas, il faut changer le filtre antibactérien.

Suivez les instructions d'utilisation du Servoport 3000 !

