SERVOX®

Tracheo Tape M duo

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REF XXXXX

PZN xxxxxxxx

(xx)xxxxxxxxxxxxxx
(xx)xxxxxx
(xx)xxxxxx

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GB Intended purpose | indication | description

The SERVOX® Tracheo Tape M duo is used in laryngectomised and tracheotomised patients at home and in a clinical setting together with tracheal cannulas or other aids with eyelets which keep the trachea open.

The SERVOX® Tracheo Tape M duo secures the aid in the tracheostoma and prevents it from accidentally slipping out.

- Reverse side of polyester tube holder with cotton coating
- Suitable for all age groups depending on the neck circumference
- Single-use, do not reuse (risk of infection)
- The maximum duration of use is 48 hours, then dispose of in household waste
- Can be seamlessly adjusted to the circumference of the neck
- Soft padding to prevent decubitus

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GB Application

- Open all Velcro fastenings.
- Slide the shorter end of the band through one of the eyelets on the neck flange from below and close the Velcro.
- Slide the longer end of the band through the other eyelet on the neck flange from below and close the Velcro.
- Place the longer part of the band around the neck.
- Connect both parts of the band with the Velcro.
- Use the Velcro to adjust the length of the band.
- Check fit. The band must be tight enough to ensure it does not accidentally come loose but not so tight that it constricts the throat!
- Cut off any excess band.
- To remove, release the Velcro and take off the band.

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GB Please note

- Regularly check the fit and adjust where necessary.
- Remeasure regularly the neck circumference regularly to ensure the correct size is always used.
- The gap between the padding and the neck flange should be as close as possible.
- Please inform us and, if necessary, the competent authority if life-threatening illness or injury occurs in connection with this product or if inpatient treatment results from its use.

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