SERVOX® Kombi-Tracheokomresse AL standard, gelocht

Orbisana
GESUNDHEITSWELT

EN	SERVOX® Combi Tracheal Compress AL, standard, with punched hole
FR	Comprese trachéale combinée AL, standard, perforée
IT	Compressa tracheale combinata AL, standard, perforata
ES	Compresas traqueales combinadas AL, estándar, perforadas
BG	SERVOX® Комбиниран трахеален компрес AL, стандартен, с отвор
HU	SERVOX® standard kombinált légszö-sebpárna AL, lyukas
EL	Συνδυαζόμενο επίθεμα τραχειοστομίας SERVOX® Kombi AL, στάνταρ, με οπή
HR	Kombinirana trahealna kompreza SERVOX®, AL, standardna, perforirana
NL	SERVOX® Combi-tracheokompres AL standaard, geperforeerd
PT	Compressa traqueal combinada SERVOX® AL standard, perfurada
PL	Kombinowany kompres tracheostomijny SERVOX® AL standard, z otworami
SL	Kombinirana standardna trahealna kompreza z luknjo SERVOX® AL

DE	Nicht wiederverwendbar – Infektionsgefahr
EN	Do not reuse – risk of infection
FR	Ne pas réutiliser – risque d'infection
IT	Non riutilizzare – rischio di infezione
ES	No reutilizar – riesgo de infección
BG	Да не се използва повторно – опасност от инфекция
HU	Ne használja újra – fertőzőkérés
EL	Μη χρησιμοποιείτε ξανά – γιατί θα είναι κινδύνος μάλιστας
HR	Nemojte ponovno upotrijebiti – većiznai rizikov, pljesavoj
NL	Niet hergebruiken – risico op infecties
PT	Nie używać powtórnie – ryzyko infekcji
PL	Izdelek ni primeren za ponovno uporabo – občastja nevarnosti okužbe

yyyy-mm

UDI

(xx)xxxxxxxxxxxx
(xx)xxxxxx
(xx)xxxxxx

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EN

Intended purpose SERVOX® Combi Tracheal Compress AL, standard, with punched hole provide cushioning for the neck flange and absorb moisture that secretes from the tracheostoma to prevent skin irritation and infections.

Indication Tracheotomised or laryngotomised tracheal cannula users with heavy secretions.

Description The wipeable, impermeable layer protects clothing from becoming soaked in case of heavy secretions. The punched hole ensures a secure fit and prevents slipping. The aluminum Coating helps prevent the compress from sticking to the skin. The punched hole ensures a more secure fit and prevents slipping.

Application Slide the tracheal cannula through the punched hole of the compress (with the aluminium coating to the tracheostoma) until the neck flange is well cushioned. Then insert the tracheal cannula as usual.

To remove, pull out the tracheal cannula as usual, pull the compress off the cannula tube and dispose of it. - **Changing frequency:** with every cannula more frequently as needed, at least every 24 hours. - If the skin is irritated, use a compress with no aluminium coating and consult a doctor.

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